AT51122

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Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, cautions and recommendations, please contact your health care professional or dealer to avoid product damage and personal injury.

CAUTION: Inspect all parts for damage caused in transit. If you notice such damage, DO NOT use the product. Contact the manufacturer for more information.

APPLICATION/INDICATIONS

The round shower stool is a medical device intended to provide a stable and easy seat support for people with disabilities using the shower.

- The shower stool provides a stable place to sit, which is very helpful for people who have difficulty standing or maintaining balance for long periods of time.
- Reduces the risk of slipping and falling in the shower, which is key to preventing bathroom accidents.
- Allows people who cannot stand for long periods of time to wash themselves comfortably without having to stand.
- For people who are recovering from surgery or injury, the stool can help them bathe independently without too much effort.

CONTRAINDICATIONS

Physical or mental limitations (such as visual impairment) that prevent safe handling.

- If the stool does not stand steadily or has not been properly secured, it can lead to slips or falls.
- In very small shower stalls, the stool can restrict freedom of movement, which paradoxically can increase the risk of falling.

If there are open wounds or sores on the user's skin, use of the stool can lead to additional pressure on these areas, which can impede healing.

- People who have severe difficulty maintaining balance, even while sitting, may need more advanced solutions, such as shower chairs with backrests or other supportive devices.
- People who experience syncope or have circulation problems (such as low blood pressure) may face additional risks when using a shower stool, especially if there are temperature changes.

FEATURES

- Aluminum heavy-duty construction
- anti-skid feet
- comfortable, easy-to-clean, swivel seat
- step height adjustment

SPECIFICATIONS

Material: aluminum


Seat height from the floor: 35.3cm - 53cm

Seat diameter: 29.6cm

Width: 36.5cm

Total weight: 1.65 kg

Maximum user weight: 136 kg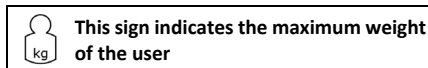

COMPONENTS:

Seat, 2 sets of legs, height-adjustable legs, 5 screws, 5 steel washers, Instruction manual.

NOTES:

In the event of a device-related "serious incident" that directly or indirectly led to, may have led to, or may lead to any of the following:

- (a) death of a patient, user or other person, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious threat to public health

the above "serious incident" must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

In case of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, consult a health care professional.

NOTE: It is forbidden to use the product other than in accordance with its intended use.

NOTE: The manufacturer is not responsible for damage caused by negligent cleaning and maintenance, or as a result of failure to follow the instructions in this manual.

TARGET PATIENT GROUP


Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this product is intended (see the section on the purpose of the product of this user manual). The product may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the device on its own or as indicated by a doctor/therapist/other specialist, the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer should be taken into account.

USE



To assemble the product, attach the legs to the underside of the seat.

Take the seat and lay it upside down on a flat surface.


Position the first set of legs under the bottom of the seat in a special groove. Then cross the second set of legs with the previous one (in the special groove on the bottom of the seat).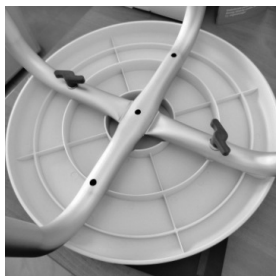

Put steel washers on each screw, then screw the screws into the holes. Make sure all the legs are immobile and stable.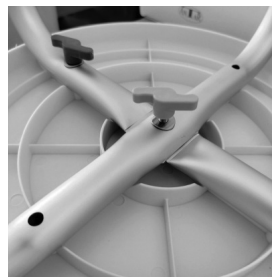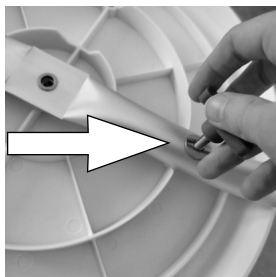

Next, adjust the legs to the desired height by pressing the latches on the stool legs and adjust the height as you prefer. Make sure that all four legs are set to the same height and that the buttons on all 4 legs are securely locked.

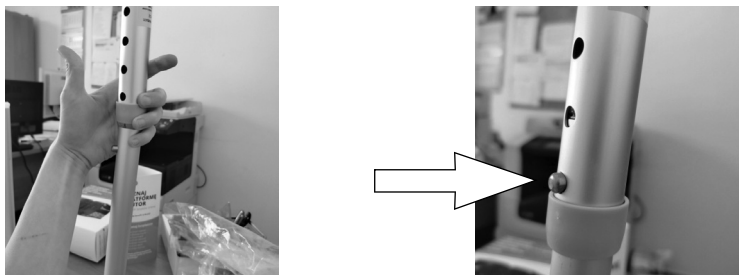Perform all disassembly steps in reverse order of assembly.

NOTE: Do not stand on the chair. Always make sure the chair stands stable. Watch out for stains and dirt accumulation, as it may affect the chair's stability and slip resistance. Do not use the chair for purposes other than those indicated.

CAUTION: When using and operating the chair and when folding and adjusting its mechanisms, there may be a danger of trapping and/or squeezing parts of the user's/companion's body in the openings/gaps between components. Perform these operations with particular care. Stabilize the position by carefully tightening the nuts/bolts after the adjustment is completed.

CLEANING AND CARE

Ordinary dirt on metal and plastic parts can be removed with standard cleaners and a soft cloth. Do not use solvents, strong detergents, avoid corrosive substances. When cleaning, do not scrub with sharp objects. It is extremely important to keep all parts of the product clean. Before each use, check that all screws are properly tightened, that no defects and mechanical damage and wear of the product are visible. Check the condition of the metal washers.

STORAGE

Store the product in a dry cool room, away from moisture and direct sunlight.

HOW TO DISPOSE OF THE PRODUCT

After the product is taken out of service, the medical device can be disposed of as ordinary municipal waste.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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