AT53042

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EN

Thank you for purchasing our product. Please read the user manual carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, notices, and recommendations, contact a healthcare professional or seller to avoid product damage or personal injury.

WARNING: Check all parts of the product for damage during transport. If such damage is found, DO NOT use the product. For more information, contact the seller.

DESCRIPTION AND FEATURES

The ankle brace is made of high-quality fabric that combines the advantages of nylon, polyester, and latex, providing excellent durability, flexibility, comfort, and a proper fit.

The brace is equipped with silicone reinforcements on the sides, which distribute the pressure from key areas to surrounding soft tissues. It provides strong and effective support for weakened muscles and joints while preventing injuries. The product is designed to offer maximum support while maintaining a comfortable fit.

It is suitable for both everyday use and sports activities. Choosing the correct size is crucial for maximizing benefits and avoiding excessive compression. If the brace is used after an injury, consult a physician if no improvement is observed within 72 hours. For severe injuries or doubts, seek medical advice immediately. For children, adult supervision is required. Keep out of reach of small children.

INDICATIONS/INTENDED USE

Ankle joint injuries

Ankle joint disorders

Rehabilitation following various ankle joint conditions

Protective support in case of weakened ankle joint

Post-injury conditions

During rehabilitation after surgeries

The brace ensures the ankle functions within its optimal, anatomical range of motion

CONTRAINDICATIONS

The brace should not be used on open wounds or infected areas, as this could worsen skin condition or spread infection.

People with severe circulatory disorders in the legs should use braces only under strict medical supervision, as overly tight braces can exacerbate circulation problems.

Patients with neuropathy may not adequately sense the pressure exerted by the brace, which could lead to further tissue damage.

Improperly fitted braces may cause discomfort, worsen existing orthopedic problems, or even cause new injuries.

In some patients, using the brace in cases of acute inflammation or significant swelling could worsen the condition.

AVAILABLE SIZES

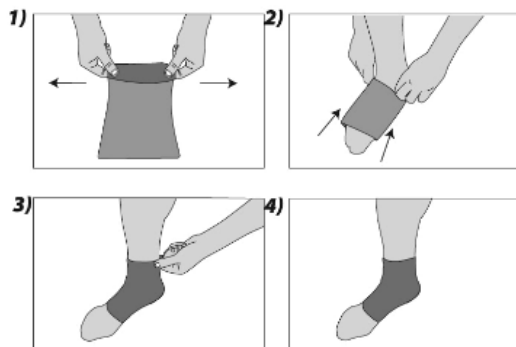
M (18–21 cm), L (21–23 cm), XL (23–26 cm), XXL (26–29 cm)

To select the correct size, measure the ankle circumference 10 cm above the joint.

USAGE

The brace is worn by sliding it onto the foot.

Composition: 60% nylon, 30% latex, and 10% Spandel

List of components: User Manual, Ankle Brace

Warnings: Do not use the product if warming liquids or ointments have been applied. Do not use on damaged skin.

Notes:

Familiarize yourself with this manual before using the brace.

Inspect the brace for seam wear before each use.

Misuse of the product may result in serious complications.

Use the brace as directed by a doctor or physiotherapist.

Do not use the brace on open wounds, abrasions, or after applying warming gels or ointments.

Wearing the brace at night is only allowed if prescribed by a doctor.

Consult a doctor regarding the duration of use.

WARNING:

In case of a "serious incident" related to the product that directly or indirectly resulted in or could result in:

- a) death of a patient, user, or another person;
- b) temporary or permanent deterioration of health; or
- c) a serious public health threat,

the "serious incident" must be reported to the manufacturer and the relevant authority in the user's country of residence. In Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices, and Biocidal Products.

WARNING:

If pain, allergic reactions, or other concerning symptoms related to the use of the medical device occur, consult a healthcare professional.

WARNING:

Using the product in ways other than intended is prohibited.

WARNING:

The manufacturer is not liable for damages caused by neglecting cleaning and maintenance or failing to follow the recommendations in this manual.

TARGET PATIENT GROUP

Individuals dealing with diseases, dysfunctions, or injuries for which this product is intended (refer to the "intended use" section). The product may be purchased independently or based on recommendations from a doctor, therapist, or other specialist. Regardless of purchase method, consider the available sizes/functions/variants, indications, and contraindications, as well as the manufacturer's information.

CLEANING AND MAINTENANCE

Hand wash the product with mild detergent at temperatures up to 40°C. Do not machine wash. Do not use strong detergents, bleach, or abrasive materials. Dry flat in the shade. Do not expose to direct sunlight. Do not tumble dry, dry-clean, or iron.

STORAGE

Store the product in a cool, dry place away from moisture and direct sunlight.

DISPOSAL

The medical device can be disposed of as regular household waste.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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