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EN

## INTRODUCTION

Thank you for purchasing our product. Before using it, please read this user manual carefully. Incorrect use may lead to damage to health, injury or accidents. Thus, in order to use the product safely and successfully, please read these instructions carefully.

### CAUTION:

In the event of a device-related "serious incident" that has directly or indirectly led to, could have led to or may lead to any of the following:

- (a) death of a patient, user or other person, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious risk to public health

the above 'serious incident' must be notified to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

### NOTES:

In case of pain, allergic reactions or other disturbing symptoms unclear to the user related to the use of the medical device, consult a healthcare professional.

### CAUTION:

The manufacturer is not responsible for damage caused by negligent cleaning, use or as a result of not following the instructions in this manual.

**ATTENTION:** It is forbidden to use the product other than for its intended purpose!

**CAUTION:** flammable product.

### PURPOSE

- knee injuries,
- ligament and patella disorders,
- rehabilitation after various knee joint disorders,
- protective measure in case of knee weakness,
- post-traumatic changes,
- rehabilitation after surgery.

### CONTRAINDICATIONS

Physical or mental limitations (e.g. visual impairment) that prevent safe handling.

**NOTE:** If soreness or rash occurs, discontinue use immediately and seek medical advice

**SIZE:** universal (knee circumference up to 39 cm), XL (knee circumference 40-50 cm)

**ORTHOSIS LENGTH:** 26 cm

**SUPPLY RANGE:** brace, instructions for use

### TARGET GROUP OF PATIENTS

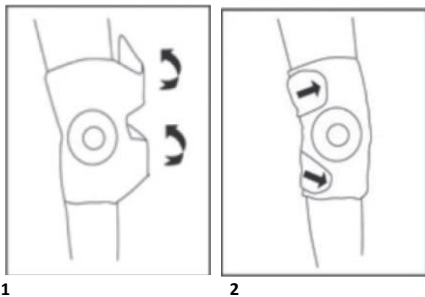
Persons suffering from diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this product is intended (see the section on intended use of the product in this manual). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Either when purchasing the device independently or when indicated by a doctor/therapist/other specialist, the available sizes/necessary functions/sizes/variants of the device, the indications and contraindications for use, as well as the information provided by the manufacturer, must be taken into account.

### Instructions for fitting:

1. the brace should be put on at the correct height so that the silicone reinforcement is at the level of the patella. The brace should be worn on the straight leg. (Fig. 1)

Fasten the Velcro strap to the right side for a snug fit and to stabilise the knee without restricting movement. (Fig. 2)

Fit the brace with the long side of the Velcro tab above the knee.

### STORAGE

The product should be stored in a dry cool place, away from moisture and direct sunlight.

### CLEANING/MAINTENANCE

The product should be hand washed with a mild detergent (e.g. grey soap) at up to 40°C. Do not iron, tumble dry, dry clean or use bleach.

### DISPOSAL

Dispose of municipal waste in accordance with the applicable disposal conditions.

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Model: .....  
Serial number: .....

Point of sale stamp

and a legible signature of the Seller: .....

1. AN TAR Sp. J., 03-068 Warsaw, ul. Zawiślńska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period AN TAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. AN TAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to AN TAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

 AN TAR Sp. J.  
03-068 Warsaw, ul. Zawiślńska 43  
Tel. 22 518 36 00, Fax 22 518 36 30  
www.antar.net

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