AT53052, AT53053, AT53054

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Introduction

Thank you for purchasing our product. Before using, please read this user manual carefully. Incorrect use may lead to health detriment, injury or accidents. Therefore, in order to use the product safely and successfully, please read this manual carefully. After purchasing the orthosis, make sure that the size of the product is correct. Otherwise, a too tightly fastened product may cause local tension, pressure on blood vessels and nerves. The first application and use of the product must be performed by a trained professional. The manufacturer is not responsible for any problems resulting from the use of incorrectly selected products and improper use.

AT53052 LemboMove™ BASIC

PURPOSE / FEATURES OF THE ORTHOSIS

The spine brace improves the stabilization of the spine in the lumbar region, reducing pain and improving the patient's daily comfort. Stabilization is ensured by elastic fastening belts and profiled reinforcements (stays), which relieve the paraspinal tissues by transferring forces that load the structures of the spine, which prevents harmful changes in the biomechanics of the lower parts of the spine. Due to its anatomical structure, the brace fits perfectly to the body and can be worn unnoticed under clothing. High-quality Velcro fasteners ensure easy and precise putting on of the orthosis and reliable fixation.

Height: 27 cm

INDICATIONS:

- Conservative / postoperative treatment
- Pain in the lumbar region
- Mild instability of the lumbar spine
- Osteochondrosis
- Spondylolisthesis
- Spondylolysis

AT53053 LemboMove™ DUO

PURPOSE / FEATURES OF THE ORTHOSIS

The spine brace improves the stabilization of the spine in the lumbar region, reducing pain and improving the patient's daily comfort. Stabilization is ensured by elastic fastening belts and profiled reinforcements (stays), which relieve the paraspinal tissues by transferring forces that load the structures of the spine, which prevents harmful changes in the biomechanics of the lower parts of the spine. Due to its anatomical structure, the brace fits perfectly to the body and can be worn unnoticed under clothing. The orthosis is equipped with a removable rear reinforcement, which provides additional support for the lumbar section and, depending on the patient's needs, can be fixed at an appropriately selected level. High-quality Velcro fasteners ensure easy and precise putting on of the orthosis and reliable fixation.

Height: 27 cm

INDICATIONS:

- Conservative / postoperative treatment
- Pain in the lumbar region
- Spondylolisthesis
- Spondylolysis
- Discopathy
- Osteoarthritis of the lumbar spine

AT53054 LemboMove™ PRO

PURPOSE / FEATURES OF THE ORTHOSIS

The spine brace has been designed to improve patient comfort at the highest possible level. Flexible straps are the basis for the stabilization of the spine in the lumbar region, and the height of the orthosis, extended to 30 cm, provides even better stabilization and functionality during use. Contoured reinforcements (stays) relieve the paraspinal tissues by transferring forces that load the structures of the spine, which prevents harmful changes in the biomechanics of the lower parts of the spine. Additional, double fastening straps and the anatomical structure of the orthosis enable excellent stabilization of the spine, perfectly adjusting to the body. High-quality Velcro fasteners ensure easy and precise putting on of the orthosis and reliable fixation. By using additional fixing straps made of stretchable material, we can achieve even better spinal stabilisation.

Height: 30 cm

INDICATIONS:

- Conservative / postoperative treatment
- Pain in the lumbar spine
- Mild instability of the lumbar spine
- Osteochondrosis
- Spondylolisthesis
- Spondylolysis

CONTRAINDICATIONS

Physical or mental limitations (e.g. visual impairment) that prevent the safe handling of the product. If soreness or a rash occurs, discontinue use immediately and consult a physician.

USE:

1. The orthosis should be put on in a standing position.
2. Place the left side of the brace on your stomach, then attach the right side to it with Velcro. Make sure that the brace is compressed enough so that it does not fall off (for additional cuffs, do the same). The possibility of tensioning the elastic straps with a Velcro end will ensure appropriate pressure adjustment.
3. Loosen and remove in the reverse order.

Warning: do not use the product if liquids or warming ointments have been used, do not use in the event of skin damage! Daily use and duration of use should be agreed with a doctor, recommended use time 3-4 hours / day. Long-term use of the orthosis can weaken the muscles.

PRODUCT CLEANING AND MAINTENANCE INSTRUCTIONS:

- The brace may only be washed by hand in lukewarm water (at 30-40°C) with a mild detergent (eg gray soap). Before taking out, be sure to remove the stays. In order to maintain the adhesion of Velcro fasteners, they should be closed before washing.
- Do not dry clean.
- Do not use bleach.
- Protect against moisture.
- Do not expose to high temperature or sunlight.
- Do not tumble dry.
- Dry when unfolded, do not dry in a hanging position.

Note: The product may only be used again when it is completely dry.

STORAGE

The product should be stored in a dry and cool place, away from direct sunlight and moisture. It is recommended to store in the original packaging

SIZES (in order to choose the right size, measure the waist circumference).

S: 75-85 cm, M: 85-95 cm, L: 95-105 cm, XL: 105-115 cm, XXL: 115-125 cm, XXXL: 125-135 cm

NOTE: In the event of a product-related „serious incident“ that led, directly or indirectly, could or could have led to any of the following events:

- a) death of a patient, user or other person or
- b) temporary or permanent deterioration in the health of a patient, user or other person, or
- c) a serious threat to public health

this „serious incident“ must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTE: In the event of pain, allergic reactions or other disturbing, unclear for the user symptoms related to the use of a medical device, please consult a healthcare professional.

ATTENTION: it is forbidden to use the product other than in accordance with its intended purpose

ATTENTION: The manufacturer is not responsible for damage caused by neglect of maintenance, inadequate servicing or resulting from failure to comply with the recommendations contained in this manual.

NOTE: If the doctor recommends using the preparation at night, make sure that it does not interfere with blood circulation; loosen the belt in case of any numbness and remove it if necessary. In case of persistent discomfort, consult a doctor.

TARGET PATIENT GROUP

Persons suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see section on intended use of the device in this instruction manual). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions and variants of the device, the indications and contraindications for use and the information provided by the manufacturer.

DISPOSAL OF THE PRODUCT

The product must be disposed of in accordance with the applicable disposal conditions.

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**WARRANTY CARD**

Modell:
 Serial number:

Point of sale stamp
 and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the rollator, starting from the date of purchase by the Buyer. Products financed by the National Health Fund are covered with the extended warranty period, which covers half of the useful life specified in the Regulation on Guaranteed Services, provided that the useful life is specified in the Regulation in months or years.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, send it immediately to ANTAR (as specified in point 1) or contact the point of sale.
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum permissible weight of the user as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts subject to normal wear and tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED. IN ORDER TO EXTEND WARRANTY PERIOD FOR PRODUCTS FINANCED BY THE NATIONAL HEALTH FUND, ATTACH A COPY OF THE ORDER FOR THE SUPPLY OF MEDICAL DEVICES BEING ORTHOPAEDIC ITEMS AND AIDS.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
 03-068 Warsaw, ul. Zawisłańska 43
 Tel. 22 518 36 00, Fax 22 518 36 30
 www.antar.net

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