**AT53521**

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Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, cautions and recommendations, please contact your health care professional or dealer to avoid damage to the product and body injury.

**NOTE:** Inspect all parts for damage caused in transit. If you notice such damage, DO NOT use the product. Contact the manufacturer for more information.

**APPLICATION**

The product is intended for wheelchair users who want to protect their body from the weather(sun rays, wind). The cover has a warming and protective function.

**CONTRAINDICATIONS**

Physical or mental limitations (such as visual impairment) that prevent safe handling. When the wearer feels uncomfortable in the warming cap, when the cap is too large or too small, tight, so that it impedes proper function, hinders vision, hearing and movement.

**FEATURES**

- capture
- 2-button closure
- extended front of the cap, which protects the legs
- shortened back of the cap
- easy to put on and take off
- Material: 100.00% polyester
- Material type: fleece

**SIZE**

Universal size  
Back length: 62 cm

**COMPONENT PARTS**

Wheelchair warming cover, instruction manual.

**NOTES:**

In case of pain, allergic reactions or other disturbing, unclear to the user symptoms associated with the use of the product, consult a health care professional.

**NOTE:** It is forbidden to use the product other than in accordance with its intended use.

**ATTENTION:** the manufacturer is not responsible for damage caused by negligent cleaning, or resulting from failure to follow the recommendations contained in this manual.

**USE**

The warming cap should be slipped over the head, then make sure that no part of the material has curled in such a way as to cause discomfort in use. Care should be taken to ensure that the warming cap does not impede comfortable movement in a wheelchair, and special care should be taken to prevent the product from coming into contact with the wheels of the wheelchair. If necessary, a hood can be placed over the head.

**CLEANING**

Wash the product at up to 40 degrees C, do not bleach or chlorinate, do not tumble dry, iron at 110 degrees C, dry clean without using solvents or strong detergents.

	Hand wash at up to 40°C
	Do not bleach
	Iron at a maximum temperature of 110 degrees celcius.
	Do not tumble dry
	Do not dry clean

**STORAGE**

Store the product in a dry cool room, away from moisture and direct sunlight.

**HOW TO DISPOSE OF THE PRODUCT**

After the device is taken out of service, the medical device can be disposed of as ordinary municipal waste

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WARRANTY CARD

Model: .....  
Serial number: .....

Point of sale stamp  
and a legible signature of the Seller: .....

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.  
03-068 Warsaw, ul. Zawiańska 43  
Tel. 22 518 36 00, Fax 22 518 36 30  
www.antar.net

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