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Model:

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Reusable waterproof pad

AT05001, AT05002, AT05003, AT05004, AT05005, AT05006

Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product before reading and understanding this manual. If you do not understand the warnings, cautions and recommendations, please contact your health care professional or dealer to avoid product damage or personal injury.

CAUTION: Inspect all parts for damage caused in transit. If you notice such damage, DO NOT use the product. See your dealer for more information.

TARGET PATIENT GROUP

Persons struggling with diseases, dysfunctions or injuries, for the treatment/rehabilitation or compensation of which this product is intended (see the section on the purpose of the product of this manual). The product may be purchased by the user independently or on the recommendation of a physician, therapist or other specialist. Whether purchasing the product on its own or as indicated by a doctor/therapist/other specialist, the available sizes/necessary functions and variants of the device, indications and contraindications for use, as well as information provided by the manufacturer, should be taken into account.

CONTRAINDICATIONS

Physical or mental limitations (e.g., visual impairment) that prevent safe handling of the product.

LIST OF COMPONENTS

Waterproof pad, instruction manual

CAUTION:

In the event of a device-related “serious incident” that directly or indirectly led to, may have led to, or may lead to any of the following:

- (a) death of a patient, user or other person, or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious threat to public health

the above “serious incident” must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

In case of pain, allergic reactions or other disturbing, unclear to the user symptoms related to the use of the medical device, consult a healthcare professional.

Extremely practical waterproof underlay that is a combination of terry cloth on the top side of the product and oilcloth on the bottom. It perfectly protects the mattress from getting dirty and wet. It is made of materials inert to the skin and respiratory system. It is used for people immobilized by illness for a long time in bed. It is used in the prevention of bedsores. The pad provides an anti-allergic barrier for the patient due to the fact that it is waterproof. It absorbs urine, moving it away from the patient's body.

WASHING INSTRUCTIONS:

1. clean using a damp sponge with detergent.
2. Maximum washing temperature of 90°C.

3. do not chlorinate! (Do not use free chlorine emitting compounds for bleaching).

4. do not dry clean!

5. Acceptable tumble drying (drying at normal temperature).

6. do not iron! (do not allow contact with heated surfaces such as a radiator).

7) Deterioration of visual-aesthetic properties (surface wrinkling type) does not affect the functional parameters of the material (physical-mechanical properties).

8. the membrane of the underlay is waterproof.

Dimensions:

AT05001 - 70 x 140 cm

AT05004 - 100 x 140 cm

AT05005 - 200 x 140 cm

AT05006 - 200 x 90 cm

NOTES AND WARNINGS:

1. It is forbidden to use the product on children under 3 years old.

2. avoid contact of the product with fire. The product may catch fire.

CAUTION: The pad does not fully protect against the formation of bedsores in people who remain in a motionless lying position for a long period of time (e.g. paralyzed people, post-stroke, unconscious people, etc.).

HOW TO DISPOSE OF THE DEVICE AFTER DECOMMISSIONING

After the device is taken out of service, the medical device can be disposed of as ordinary municipal waste except for electrical products - proceed in a manner appropriate for the disposal of electrical and electronic equipment.

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Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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