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
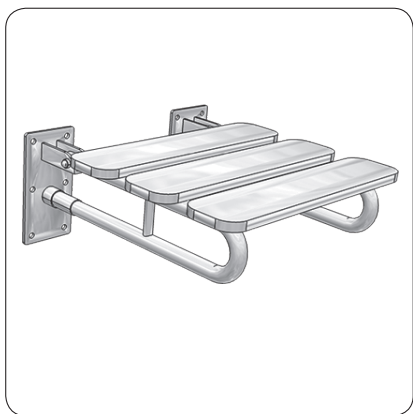
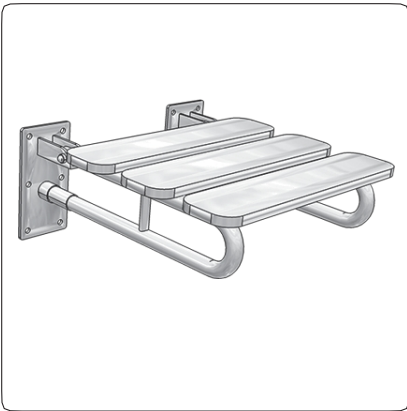
INSTRUCTIONS FOR USE AND INSTALLATION OF BATHROOM

SEATS FOR THE DISABLED:

SHOWER SPU-370, BATHTUB SNW-500

SPU-370

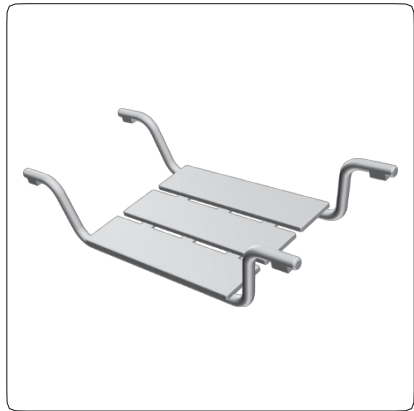
SNW-500

Bathroom seats help people with limited mobility to maintain daily personal hygiene and contribute to a significant increase in safety when moving around in the shower or bath.

METHOD OF INSTALLATION

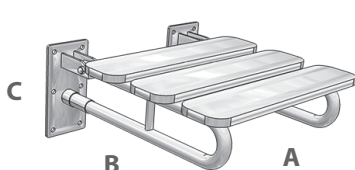
I. BATHROOM SEAT FOR HANDICAPPED SHOWER SPU-370 - FLOOR FIXING

	A	B	C
	[mm]	[mm]	[mm]
SPU-370	370	400	200

1. It is recommended to install the seat in a horizontal position.
2. Drill holes for connector bodies with a diameter corresponding to the diameter of the connector body, the location of which in the wall will correspond to the location of the holes in the base of the shower seat. The depth of the hole in the base should be equal to the seating depth of the fastener increased by 10 mm.
3. Clean the hole of dust.
4. Insert the connector body into the drilled holes.
5. Put pads made of plastic between the ground and the bases of the shower seat.
6. Screw in, steel screws into the connector bodies for their full length. For the screws included with the bracket, use a substrate made of solid ceramic brick and silicate or ordinary concrete. Make sure that the hole does not fall in the grout. It is not recommended to install the seat on a substrate made of plasterboard.
7. Put finishing plugs on the heads of steel screws.
8. After decommissioning, the product can be treated as municipal waste.

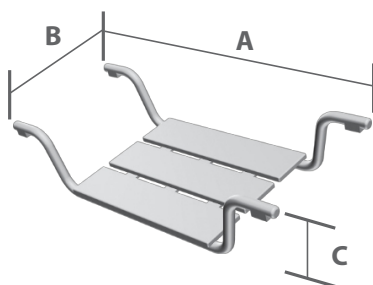II. BATHROOM SEAT FOR THE DISABLED CONVERTIBLE SNW-500 - BATHTUB MOUNT

	A	B	C
	[mm]	[mm]	[mm]
SNW-500	690	290	200

1. It is recommended to fix the seat in a horizontal position on the bathtub, so that the arms of the seat find their support on the edges of the bathtub.
2. After decommissioning, the product can be treated as municipal waste.

CONSERVATION

During the use of the shower seat, periodically check the fixing pins for looseness. If any looseness is found, check the cause of the looseness and tighten the screws or replace with new ones along with the connector bodies. When taken out of service, the product can be treated as municipal waste.

DISINFECTION

The bathroom seat is a reusable product. In order to maintain proper hygiene after each use, the product should be sprayed with an aerosol preparation, a product made on the basis of sodium hypochlorite, and left for a period of 15 minutes. Then rinse thoroughly with running water.

WARNINGS

1. After installing the shower seat, make sure that it is well fixed to the ground and does not risk falling off. If the seat is too poorly attached, it risks falling off and injuring the user.
2. It is forbidden to install the shower seat to weak walls, such as drywall, as the wall may collapse during use, and the user may suffer severe injuries as a result.

THE PERMISSIBLE STATIC LOAD IS 100 KG

THIS SIGN INDICATES THE MAXIMUM WEIGHT OF THE USER

ATTENTION:

In the event of a product-related "serious incident" that directly or indirectly led to, may have led to, or may lead to any of the following:

- a) death of the condition of a patient, user or other person or
- b) temporary or permanent deterioration in the health of a patient, user or other person or
- c) a serious threat to public health

the above "serious incident" should be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

If you experience pain, allergic reactions, or other disturbing symptoms that are unclear to you related to the use of the medical device, consult a healthcare professional.

HOW TO DISPOSE OF THE PRODUCT AFTER DECOMMISSIONING

Once the device is decommissioned, the medical device can be disposed of like regular municipal waste

WARRANTY CARD

Model:

Series Number:

Seal of the point of sale and legible signature of the Vendor:

- 1) ANTAR Sp. J., 03-068 Warsaw, Zawisłańska 43 St., hereby provides a 12-month warranty for the product, starting from the date of purchase of the product by the Buyer. For products, the purchase of which is reimbursed by the National Health Fund, the warranty period is extended and covers half of the period of use specified in the Regulation on Guaranteed Services, provided that the period of use is specified in the Regulation in months or years.
- 2) During the warranty period, ANTAR undertakes to perform the repairs necessary to restore the proper functioning of the product, within a period of 14 days from the date of receipt of the product by the service along with the proof of purchase and the warranty card.
- 3) ANTAR reserves the right to replace the product if it is determined that the repair costs are not cost-effective. Repair or replacement of defective products does not extend the warranty period.
- 4) If the product is found to be malfunctioning, it should be sent immediately to ANTAR's address (given in Section 1) or contact the point of sale.
- 5) Before use, read the instructions carefully and follow them. Use of the product contrary to its intended use and recommendations will result in irrevocable loss of warranty.
- 6) The warranty does not cover damage caused by external mechanical force, loading in excess of the user's maximum permissible weight as described in the manual, and contact of the product with alcohol, oily substances or gasoline. The warranty does not cover parts subject to normal wear and tear during use, such as the bearing, rubber parts and others.
- 7) The product submitted to the service in connection with the complaint must not cause an epidemic.
- 8) THIS WARRANTY IS VALID ONLY WITH PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A CLAIM, THE PROOF OF PURCHASE MUST BE ATTACHED. IN ORDER TO RECOGNIZE THE EXTENDED WARRANTY PERIOD FOR PRODUCTS THE PURCHASE OF WHICH WAS REIMBURSED BY THE NATIONAL HEALTH FUND, A COPY OF THE ORDER FOR THE SUPPLY OF MEDICAL DEVICES THAT ARE ORTHOPEDIC OBJECTS AND AIDS MUST BE ATTACHED.
- 9) Claims submitted without proof of purchase and warranty card with product serial number will not be accepted.

ANTAR Sp. J.
 03-068 Warsaw, 43 Zawisłańska St.
 Poland
 Tel. 22 518 36 00, Fax 22 518 36 30
 www.antar.net e-mail: antar@antar.net

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