**INSTRUKCJE UŻYTKOWANIA I MONTAŻU UCHWYTÓW DLA
NIEPEŁNOSPRAWNYCH**
INSTRUCTIONS FOR USE AND ASSEMBLY OF WHEELCHAIR HOLDERS
**GEBRAUCHSANWEISUNG UND MONTAGE VON SICHERHEITSGRIFFEN FÜR
DAS BAD (FÜR PERSONEN MIT MOBILITÄTSEINSCHRÄNKUNGEN, ÄLTERE
MENSCHEN ODER ANDEREN GESUNDHEITLICHEN EINSCHRÄNKUNGEN)**
NÁVOD K POUŽITÍ A MONTÁŽI DRŽÁKŮ PRO ZDRAVOTNĚ POSTIŽENÉ
**NÁVOD NA POUŽITIE A MONTÁŽ DRŽIAKOV PRE ZDRAVOTNE
POSTIHNUTÝCH**
MODE D'EMPLOI ET MONTAGE DES SUPPORTS POUR HANDICAPÉS
**INSTRUCTIES VOOR GEBRUIK EN MONTAGE VAN
GEHANDICAPTENHOUDERS**
**INSTRUCCIONES DE USO Y MONTAJE DE LOS SOPORTES PARA
DISCAPACITADOS**
ISTRUZIONI PER L'USO E IL MONTAGGIO DEI SUPPORTI PER DISABILI
BRUKSANVISNING OCH MONTERING AV HANDIKAPPHÅLLARE

MD

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EN

Thank you for purchasing our product. Please read the instructions for use carefully. Do not use the product until you have read and understood these instructions. If you do not understand the warnings, cautions and recommendations, please contact your healthcare provider or dealer to avoid damage to the product or injury.

NOTE: Check all parts for transportation damage. If you find such damage, DO NOT use the product. Contact the manufacturer for further information.

THE USER MANUAL APPLIES TO THE FOLLOWING PRODUCTS: U-200, U-300, U400, U-500, U-600, U-700, U-800, U-900, UKL-800/400, UKL800/400SN, UKL-1000/400, UKP 800/400, UKP-800/400 SN, UKP-1000/400, US-600/120°, US-600/120°SN, US-600/150°, US-600/30°, US-600/60°, US-600/90°, US-600/90°SN, UU-500, UU-600, UUCH-500, UUCH-500SN, UUCH-600, UUCH-600SN, UUCH-700, UUCH-700SN, UUCHW-500, UUCHW-500SN, UUCHW-600, UUCHW-600SN, UUCHW-700, UUCHW-700SN, UUL-700, UUL-700SN, UUP-700, UUP-700SN, UWCL-700/700, UWCP-700/700

INDIKATION

The handle for the disabled supports the balance and mobility of people with limited mobility and is suitable for installation in the bathroom or next to the bed. The handle is suitable for use both at home and in care facilities.

INSTALLATION METHOD

1. drill holes for the fixing elements with a diameter that corresponds to the diameter of the fixing element, whereby the position of the holes in the wall must match the position of the holes in the base of the bracket (as shown in the drawing). The depth of the hole in the base should correspond to the anchoring depth of the fixing element plus 10 mm.
2. clean the hole to remove dust.
3. insert the plug body into the drilled holes.
4. place plastic washers between the substrate and the handle.
5. screw full-length steel screws into the connection bodies. A substrate made of solid ceramic and silicate bricks or normal concrete must be used for the screws. Care must be taken to ensure that the drill hole does not fall through the grout. It is not permitted to install the bracket on a plasterboard substrate.
6. place the end caps on the heads of the steel screws.
- 7 After decommissioning, the product can be treated as municipal waste.

METHOD OF USE

1. grasp the handle firmly with your hand. Make sure that your hand does not slip on the handle, especially if the handle is wet.
2. pull the handle upwards and perform the desired action, e.g. standing up.
- 3 Before releasing the handle, make sure that you can safely maintain the position you have assumed without the support of the handle.

MAINTENANCE

While the bath handle is in use, the fastening pins should be checked regularly for play. If play is detected, check the cause and tighten the screws or replace them with new ones together with the connecting pieces.

If the handle arm spontaneously falls out of the vertical position, the fixing screw must be tightened (applies to: UUCH-500, UUCH-600, UUCH-700, UUCHW-500, UUCHW-600, UUCHW-700).

UTYLISATION

Dispose of municipal waste in accordance with the applicable disposal regulations.

DISINFECTION

The bathroom handle is a reusable product. For good hygiene, spray the product with a sodium hypochlorite-based aerosol preparation after each use and leave it to work for 15 minutes. Then rinse thoroughly under running water.

WARNINGS:

- 1 After installing the handles, make sure that they are securely fastened to the wall and cannot fall down. If the handles are not securely fastened, there is a risk of them falling down and injuring the user.
- 2 It is forbidden to attach the brackets to weak walls, such as plasterboard, as the wall may collapse during use and the user may suffer serious injury as a result.
3. special care should be taken to ensure that the handle is gripped firmly and securely (especially if it is wet) before the body weight rests completely on the handle. Gripping the handle too weakly can cause the user to tip over and injure themselves.
4. if the product becomes deformed due to prolonged and intensive use, discontinue use and replace the product with a new one. Prolonged use of a defective product can cause it to come loose and injure the user.
- 5 For painted products, the product is well protected against corrosion. However, if rust forms during prolonged use of the product in extreme conditions (humid environment, high salt content), replace the product with a new one and consider purchasing a stainless steel product. Prolonged use of a damaged product may cause it to come loose and injure the user.

NOTE: Any serious incident involving a device must be reported to the manufacturer and the competent authority of the Member State in which the user or patient resides.

CAUTION: The manufacturer cannot be held responsible for damage caused by negligent maintenance, inadequate servicing or failure to follow the instructions in this manual.

ATTENTION: It is forbidden to use the product for purposes other than those for which it is intended!

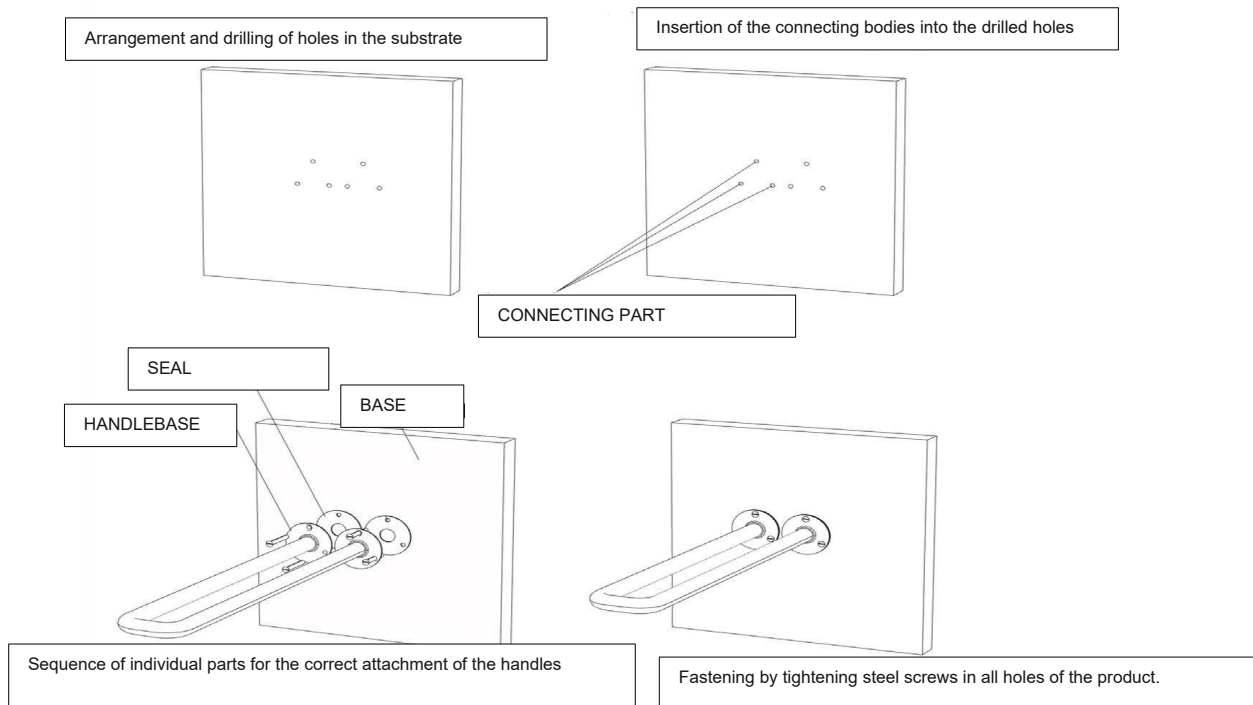
6. THE PERMISSIBLE LOAD IS 100 KG

7. only the dowels supplied may be used to install the bath handle.

This symbol indicates the maximum weight of the user.

MONTAGE

NOTES:

In the event of a "serious incident" in connection with the product which has led, could have led or could lead directly or indirectly to one of the following events:

- (a) death of a patient, user or other person; or
- (b) temporary or permanent deterioration in the state of health of a patient, user or other person; or
- (c) a serious risk to public health

The above-mentioned "serious incident" must be reported to the manufacturer and the competent authority of the Member State in which the user or patient is located. In the case of Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

In the event of pain, allergic reactions or other disturbing symptoms that are unclear to the user in connection with the use of the medical device, a doctor should be consulted.

NOTE: It is forbidden to use the product for purposes other than those for which it is intended.

ATTENTION: The manufacturer is not liable for damage caused by negligent cleaning and maintenance or by failure to observe the recommendations contained in these instructions for use.

TARGET PATIENT GROUP

Persons suffering from diseases, functional disorders or injuries for whose treatment/rehabilitation or compensation this product is intended (see section on the intended use of the product in these instructions for use). The product can be purchased by the user himself or on the recommendation of a doctor, therapist or other specialist. Regardless of whether you purchase the product yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/required functions and variants of the product, the indications and contraindications for use and the information provided by the manufacturer.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. AN TAR Sp. J., 03-068 Warsaw, ul. Zawiślńska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period AN TAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. AN TAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to AN TAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślńska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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